

# HOW TO REMOVE DISPOSABLE GLOVES SAFELY



1. Grasp the outside of the glove in the wrist area



2. Peel the glove away from your body, turn it inside-out



3. Hold the inside-out glove in the other hand



4. Slide your fingers under the wrist of the other glove



5. Peel the glove away from your body, turn it inside-out




6. Leave the first glove inside the second



7. Dispose the gloves safely



8. Wash your hands thoroughly

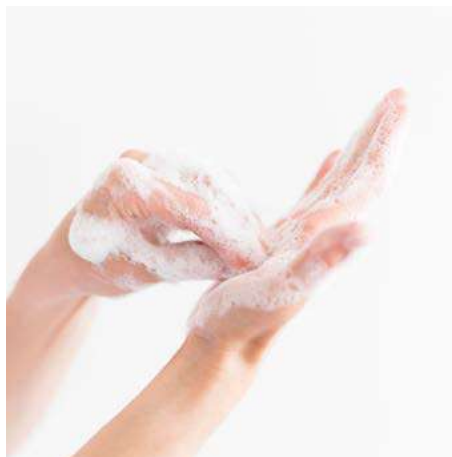

Removing gloves this way does not spread potential contaminants.

Stay safe

Protect yourself, protect others